



GMF- LD01 LED HEADLIGHT USER GUIDE



INTENDED USE

The LED headlight provides supplemental lighting for general observation.

ASSEMBLY AND OPERATION

1. Open the battery pack box and remove the battery and charging cord.
2. Plug the headlight cable into the plug on the top of the battery pack. It is recommended to route the cable through the clips on the headgear.

HEADGEAR ADJUSTMENT

The optional top adjustment (models: GMF-LD02) is very important; therefore it should be done first to gain maximum comfort. The top adjustment allows the headband to sit as low as possible on the head. Use knob at top of headgear to adjust headgear.

Then, expand the lower headgear band by turning the adjuster knob on the rear of the headgear. Place the headgear over the head and tighten the headgear around the head until snug by turning the adjuster.

HEADLIGHT OPERATION

1. Select the switch on the battery pack to the ON position.
2. Adjust the size of illumination spot by turning grooved dial on headlight.



WARNING: Do not look directly into headlight when illuminated. Eye injury may result.

HEADLIGHT ADJUSTMENT

The headlight can be adjusted up or down by tilting it up or down on the headgear pivot.

CHARGING BATTERY

Plug the charging cord end shaped like a figure 8 into the bottom of the battery pack. Plug the other end of the charging cord into a suitable wall plug. The charging indicator will illuminate. Depending on battery level, full charging may take as long as 6 hours. The headlight can be used while charging.

CLEANING AND MAINTENANCE

CAUTION: Turn the headlight system OFF and ensure charging cord is unplugged before cleaning.
DO NOT AUTOCLAVE!

General cleaning can be performed with a mild detergent and wiping cloth.

Headbands and headlight shall not be immersed into disinfecting liquids or sterilizers.

Clean the optics only with lens soft tissue. Follow the instructions on the tissue package.

PRECAUTIONS



Do not look directly into the headlight when illuminated. Eye injury may result.



Do not attempt to connect the charging cord with damp or wet hands. Electric shock may occur.

WARRANTY

Headlight

The LED Headlight System is guaranteed against all defects in materials and workmanship for 1 year.

This warranty is void if service is performed by anyone other than the manufacturer.

AVAILABLE PARTS AND COMPONENTS

Battery Pack for LED Headlight System

Order#
GMF-BC01

CONTACT INFORMATION

Gulf Fiberoptics, Inc.

448 Commerce Blvd.
Oldsmar, Florida 34677
U.S.A.

Telephone: +1 813.855.6618

Facsimile: +1 813.855.6627

E-mail: contact@gulffiberoptics.com

Website: www.gulffiberoptics.com